

# CEREDERM®

## What is Cerederm®?

Invisible and easy to use, Cerederm® is a self-adhesive silicone dressing for scars treatment.

The silicone gel relieves itching, softens the scar and reduces its redness. The medical grade silicone smoothes out the scar and limits its development.

All the components of Cerederm® dressings have been selected for their biocompatibility. Special attention has been paid to the medical grade silicone perfecting.

## For which kind of scars?

Scars often appear after a burn, a surgical operation, an accident, a skin reaction...

Cerederm® dressings are highly recommended in the preventive treatment of scars caused by surgical operations, hypertrophic scars (raised and red coloured) and keloids (fibrous excrescence, firm and thick).

The dressing is for personal use only (one patient) and must not be used by another person.

**Contraindications :** Cerederm® must not be used on open, infected or ulcerated wounds, bedsores, weeping or burns. Cerederm® must not be worn on mucous membranes either.

## How to use it?

For optimum effectiveness, Cerederm® must cover the scar by more than a centimetre. The dressing must be worn daily over a treatment period of 2 to 6 months.

The dressing intrinsic qualities, coupled with an exclusive technology, guarantee a perfect adherence to the skin and allow a painfree removal of the dressing.

Effective  
Flexible  
Invisible  
Painless



**Cerederm® is :**

**Easy to cut :** it fits the scar size.

**Flexible :** it takes on the exact shapes of the body.

**Painless :** comfortable, it can be worn 24 hours a day.

**Thin & invisible :** it is discreet.

**Self-adhesive :** it stays perfectly in position.

**Washable :** the same dressing can be worn for 2 to 3 days.

**Effective :** it is used on its own. No need to add cream.

## Instructions

**1.** Remove the dressing from its wrapping. It is possible to cut the bandage to the required size in order to cover up the scar while providing enough overlapping (at least a centimetre).



**2.** Position the Cerederm® with the adhesive side on the scar and take off gently and progressively the protective film, while fitting properly the dressing on the skin.



**3.** Use the Cerederm® dressing alone, there's no need to add cream. However, it is possible to provide an additional support (by wearing an elastic bandage or a compressive garment).



**4.** Clean the dressing once a day, using clear water and neutral soap. Clean its surface half by half, in order to put it back in place easily.



## Results

Cerederm® dressings are highly recommended in the preventive and curative treatment of hypertrophic or keloid scars caused by surgical operations.


Results are visible after two to three months use. More flexible, smoother and redless, the scar is visibly reduced.

## Advice

- It is recommended to inspect and clean the scar at least once a day with clear water and neutral soap. The dressing can be cleaned the same way (proceed half by half in order to put it back in place easily).
- The dressing must be worn daily during two to three days (ideally 24 hours a day). The same dressing must not be worn more than five days or must be replaced as soon as its adhesive properties are deteriorating.
- The dressing is strictly individual (monopatient) and must not be used by another person.
- In case of skin reaction, stop the treatment and consult the doctor.

**Caution : The dressing does not protect from UVs rays.**

### Contact :

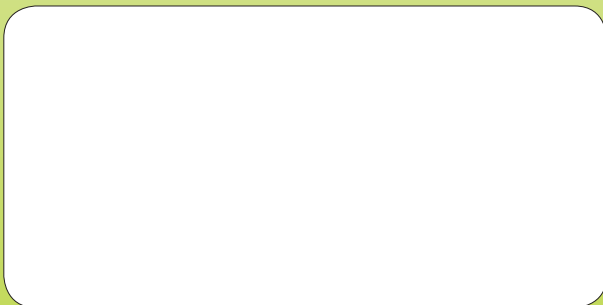


EN3-A-019-PAS -EN  
2014-04

# CEREDERM®

## Cerederm® dressings sizes

Cerederm® dressing comes in 5 models : **3 rectangular models** of different sizes and **2 specific models** : anchors and crowns, particularly suited for postoperative periods after reconstruction or augmentation mammary surgeries.

**Packaging : 10 units boxes.**

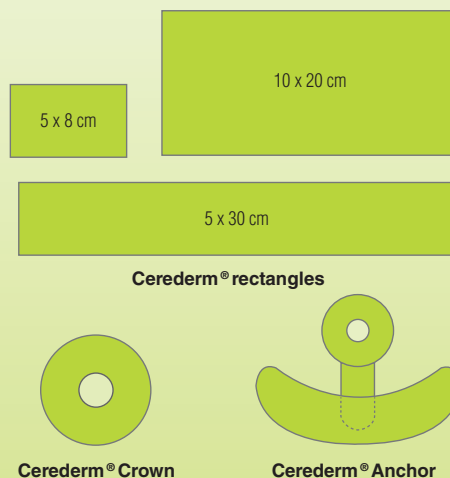

**Hanstronics Medical Equipments Pvt. Ltd.**  
1551, Aggarwal metro Heights, Netaji Subhash Place,  
Pitampura, New Delhi - 110034  
**Ph.** : +91-11-47076038, 42060250  
**Email** : hmeplhanstronics.com  
**Web** : www.hanstronics.com

## CEREPLAS

Actipôle 2 - Avenue de la Solette  
59554 Sallily lez Cambrai - FR

t : +33(0)3 27 83 69 57  
f : +33(0)3 27 83 70 42

[www.cereplas.com](http://www.cereplas.com)  
[info@cereplas.com](mailto:info@cereplas.com)



# Effective Painless Invisible

Self-adhesive silicone dressings

# CEREDERM®

Hypertrophic and keloid scars  
management & treatment

